




020 3875 8530 | **MediMatch**
www.medimatch.co.uk | **DENTAL LABORATORY**




HOW TO START SENDING YOUR SCANS TO MEDIMATCH

scan@medimatch.co.uk


Implant and trios support
T: 020 3875 8530

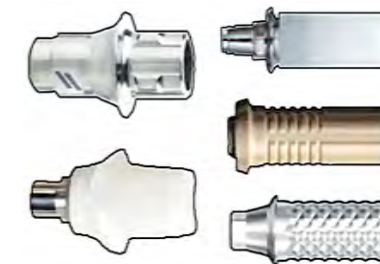


1 On your scanner Go to: **Configure -- Lab Connections -- Configure Communicate**

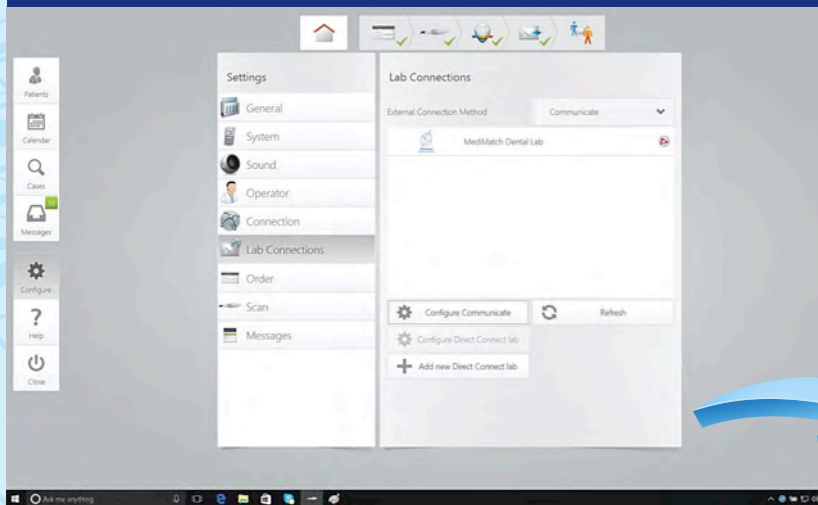


2 Add MediMatch Lab with: **scan@medimatch.co.uk**

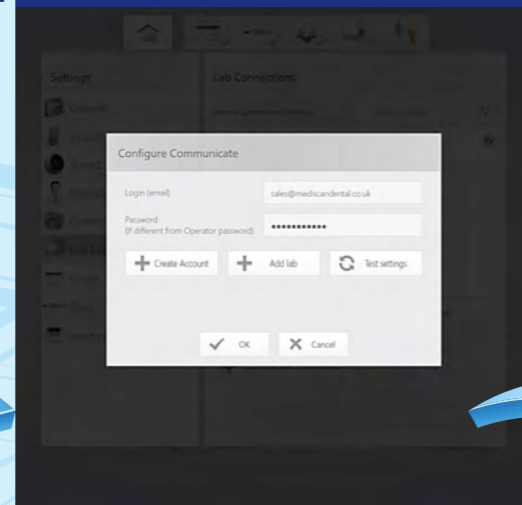


3 Log on to 3shape communicate using your own email and password

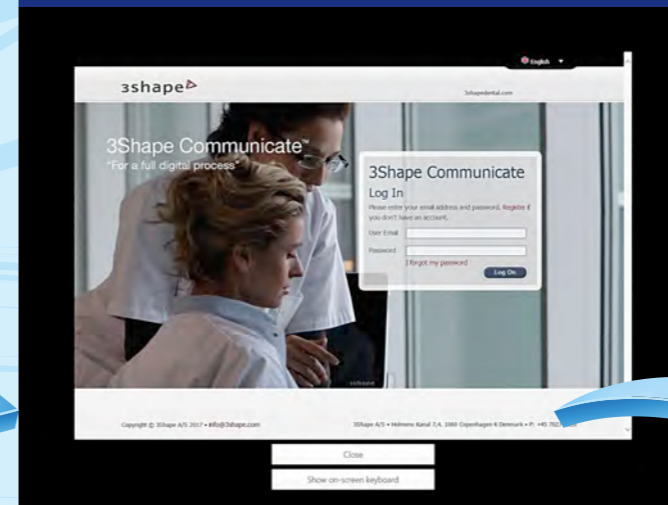

4 In 3shape Communicate go to: **Connections**

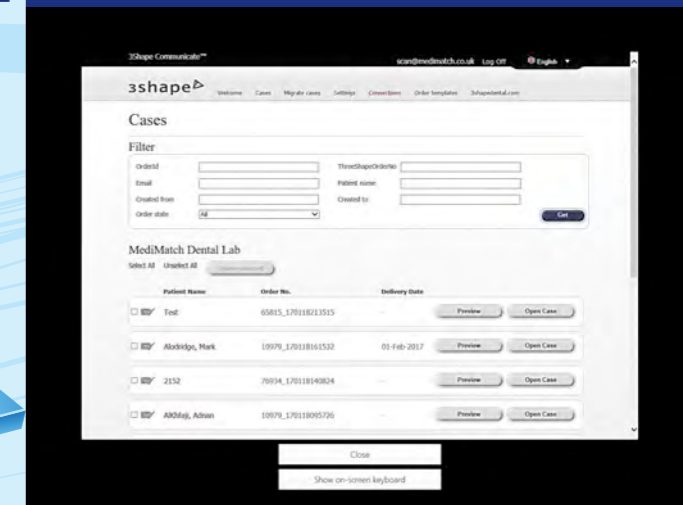


5 Add connection at the bottom of the page

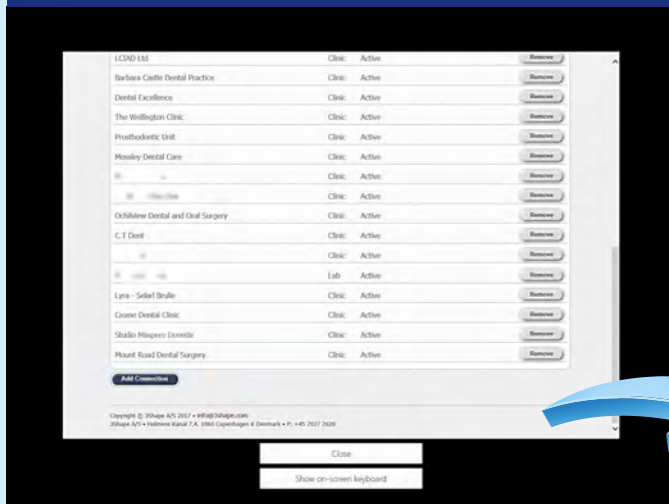

6 Enter **scan@medimatch.co.uk** and click the "Find" button



7 On your scanner, go to **Lab Connections** and refresh. Start sending your cases to MediMatch

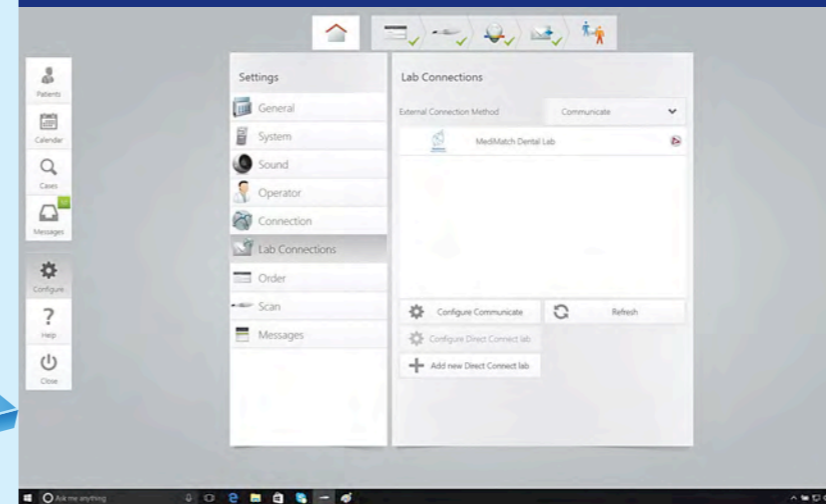

020 3875 8530
www.medimatch.co.uk



MediMatch - YOUR DIGITAL LAB